


Directions for applying the ImPad hand cover

Step 1



Apply stockinette over the hand.
Avoid wrinkles.

Step 2

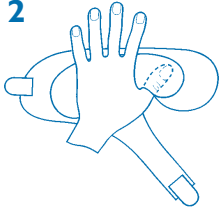

Select an ImPad.

Red graphics – *Left*. Blue graphics – *Right*.

Place the hand on top of the inflation pad as shown by the graphics on the ImPad, with the thumb located into the cutout provided.

Step 3



Wrap the ends of the cover across the back of the hand, fitting snugly and secure with the fastener tab.

Next, wrap the wrist strap around the base of the thumb and secure with the fastener tab.

Check that the hand cover is fitted snugly and the patient is comfortable.

CAUTION: FOR FULL IMPAD LIFE ONLY INFLATE WHEN FITTED TO THE HAND.