



## Reconstitution Guide



1 Peel open inner pouch.  
Remove syringes (product syringe and fluid transfer syringe).




Fill fluid transfer syringe:


Package Size	Mix w/ Sterile Saline
2 cc	1.5 cc
5 cc	3.5 cc
10 cc	7.0 cc



3 Attach fluid transfer syringe to female luer port on the product syringe and hold the assembly vertically with the product syringe on top.

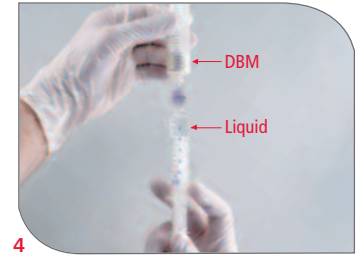

4 While holding the assembly upright, and with constant and steady pressure, inject the saline into the product syringe.

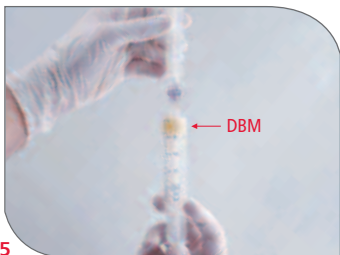

5 With the product syringe as shown, compress the plunger of the product syringe until all of the air is removed and the powder is wetted. Stop compressing if saline starts to appear on the top side of the plunger.



6 Wait 30 seconds for the saline to be absorbed into the powder.



7 Grasp the valve assembly at the end of the product syringe and twist to remove the valve and fluid transfer syringe.



8 Apply pressure to the product plunger to express the paste.



9 Mold and handle paste as desired. Discard appropriately any unused product and fluid transfer syringe.



⚠ Refer to the package insert supplied with product for specific information on contraindications, warnings, precautions, adverse reaction information, and sterilization.

Origen is a registered trademark of Blackstone Medical, Inc.



For more information contact your local representative or call 1.800.BoneFix  
www.orthofix.com