



Xtraordinary - packaged as a sterile kit in three time-saving configurations, ready for all long bone trauma applications.

Xtra light - as strong as conventional fixators, only lighter, for easier application and more patient comfort.

X-ray friendly - the first large fixator body to be radiolucent.

XCaliber - High Caliber Fixation from Orthofix.

XCaliber. **X**actly.

- Surprisingly Light Weight
- Excellent Stability
- Sterile Kit, Ready to Use

XCaliber Articulated Ankle

XCaliber Articulated Ankle

Minimally invasive, stable fixation of pilon fractures.

Ball-joints allow correction up to 44° from straight position, in any direction, with fixator in place.




Ball joints lock/unlock from either side.



One instrument for fixator locking/unlocking.

Radiolucent Ankle Pin Guide provides visualization of the rotational axis of the tibio-talar joint for correct screw positioning.

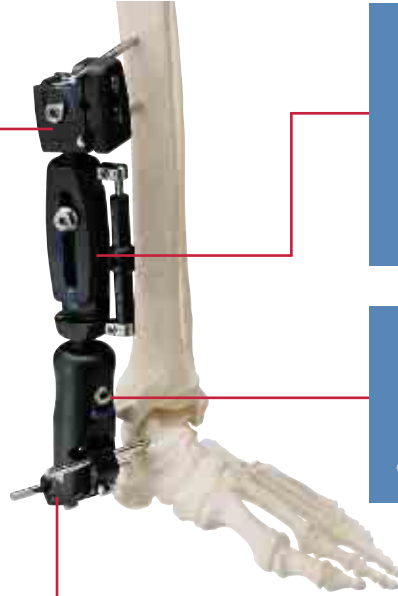

Ball-joint and telescoping body for anatomic reduction.

Distraction unit for ligamentotaxis.



Excellent stability and early joint movement with the radiolucent Articulated Ankle Clamp.

Case Study

59-year-old woman, open pilon fracture.



Minimal internal fixation of the articular surface and bridging of the joint with the XCaliber Articulated Ankle Fixator with OsteoTite bone screws.



Fixator removed at 120 days with good functional recovery.



Case provided by: Dr. M. Manca, Hospital of Massa-Carrara, Italy.




Pre-assembled and Packaged Sterile
Ready to use, straight from the box.

Ordering Information

| Part No. | Product Description |
|----------|--|
| 99-91245 | XCaliber Articulated Ankle Kit, with compression/distraction unit, Sterile |
| 99-91038 | Supplementary Screw Holder Kit, Sterile |

XCaliber Fixator Is Available In Three Configurations.

Use with XCaliber Bone Screws.

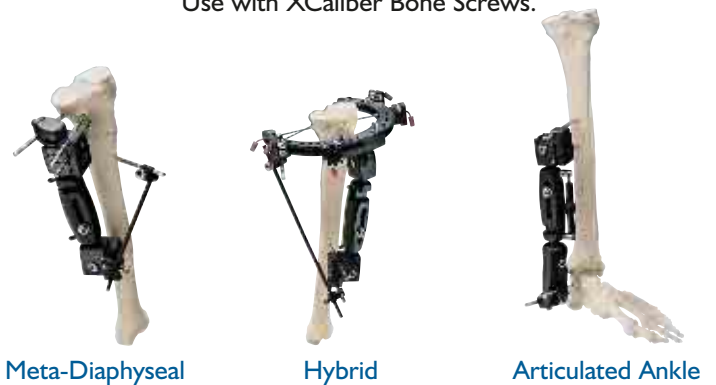

Meta-Diaphyseal

Hybrid

Articulated Ankle

Your Distributor is:

www.orthofix.com

XC-0308(A)-PL-E0 05/04

ORTHOFIX
ALWAYS INNOVATING